

CORRECT DICOM IMAGES

RESOURCE PATIENT MANAGEMENT SYSTEM (RPMS)



*****WARNING!*****

NOTICE TO USERS:

BY ACCESSING PROTECTED HEALTH INFORMATION, YOU ARE ACCEPTING ALL RESPONSIBILITIES AND CONSEQUENCES OF BOTH FEDERAL PRIVACY LAWS THAT SAFEGUARD PATIENT HEALTH RECORDS. THE RPMS CHART SENSITIVITY AND TRACKING SYSTEM ELECTRONICALLY MAINTAINS A LOG OF EVERY USER WHO ACCESSES A PATIENT'S RECORD IN ANY RPMS PACKAGE. UNAUTHORIZED USE OF THIS SYSTEM MAY SUBJECT YOU TO CRIMINAL PROSECUTION AND PENALTIES.

BE SURE TO REPORT ANY COMPUTER PROBLEMS TO THE SITE MANAGER. THIS WILL INSURE THAT ACTION WILL BE TAKEN TO RESOLVE THEM.

Volume set: NNMC:CACHE UCI: NNMC Device: /dev/pts/45 (sr-74955.d1.na.ihs.gov)

ACCESS CODE: ← **LOG INTO RPMS**

Volume set: NNMC:CACHE UCI: NNMC Device: /dev/pts/45 (sr-74955.d1.na.ihp.gov)

ACCESS CODE: *****

Good afternoon GARY

You last signed on today at 11:15

Select DIVISION: NORTHERN NAVAJO MEDICAL CTR// NAVAJO SHIPROCK
01 NM MC(M) 393601

Site set to NORTHERN NAVAJO MEDICAL CTR

DISP Display a Rad/Nuc Med Report
GIS GIS Interface Menu ...
HS Generate Health Summary
LAB Results menu ...
RAD Rad/Nuc Med Total System Menu ...
REP Imaging Site Reports ...
SCAN SCAN the patient files ...
SCH Scheduling Menu ...
VIEW View patient's registration data
WHM Women's Health Menu ...

TYPE IN DICOM

Select RADIOLOGY STAFF Option: dicom

AnzioWin Version 11.3j alt-M for HELP

NUM

- GIS GIS Interface Menu ...
- HS Generate Health Summary
- LAB Results menu ...
- RAD Rad/Nuc Med Total System Menu ...
- REP Imaging Site Reports ...
- SCAN SCAN the patient files ...
- SCH Scheduling Menu ...
- VIEW View patient's registration data
- WHM Women's Health Menu ...

Select RADIOLOGY STAFF Option: dicom Menu Options

- ECTP Edit CT PARAMETER File
- ICTP Display MAGD CT PARAMETER entries
- ECRP Edit CR PARAMETER File
- ICRP Display MAGD CR PARAMETER entries
- Correct Clinical Specialties DICOM File Entries
- Correct RAD-DICOM File Entries
- List Unread Studies
- Print Dicom Failed Image File Entries.
- Validate DICOM Correct Information

Select Dicom Menu Options Option: correct **TYPE IN CORRECT**

```
RAD   Rad/Nuc Med Total System Menu ...
REP   Imaging Site Reports ...
SCAN  SCAN the patient files ...
SCH   Scheduling Menu ...
VIEW  View patient's registration data
WHM   Women's Health Menu ...
```

Select RADIOLOGY STAFF Option: dicom Menu Options

```
ECTP  Edit CT PARAMETER File
ICTP  Display MAGD CT PARAMETER entries
ECRP  Edit CR PARAMETER File
ICRP  Display MAGD CR PARAMETER entries
       Correct Clinical Specialties DICOM File Entries
       Correct RAD-DICOM File Entries
       List Unread Studies
       Print Dicom Failed Image File Entries.
       Validate DICOM Correct Information
```

Select Dicom Menu Options Option: correct

- 1 Correct Clinical Specialties DICOM File Entries
- 2 Correct RAD-DICOM File Entries

CHOOSE 1-2: 2

SELECT CHOICE 2

ECTP Edit CT PARAMETER File
ICTP Display MAGD CT PARAMETER entries
ECRP Edit CR PARAMETER File
ICRP Display MAGD CR PARAMETER entries
Correct Clinical Specialties DICOM File Entries
Correct RAD-DICOM File Entries
List Unread Studies
Print Dicom Failed Image File Entries.
Validate DICOM Correct Information

Select Dicom Menu Options Option: correct

- 1 Correct Clinical Specialties DICOM File Entries
- 2 Correct RAD-DICOM File Entries

CHOOSE 1-2: 2 Correct RAD-DICOM File Entries

Select one of the following:

P Patient
L Loop thru file
D Specify a Date Range

Update entries by: L ← **SELECT LOOP THRU FILE**

AnzioWin Version 11.3j alt-M for HELP

NUM

Select Dicom Menu Options Option: correct

- 1 Correct Clinical Specialties DICOM File Entries
- 2 Correct RAD-DICOM File Entries

CHOOSE 1-2: 2 Correct RAD-DICOM File Entries

Select one of the following:

- P Patient
- L Loop thru file
- D Specify a Date Range

**THIS IS A BAD IMAGE
BECAUSE IT HAS A
BAD CASE NUMBER**

Update entries by: Loop thru file

*****Processing entry*****

PATIENT: TEST,TEST

SSN: 1

RADIOLOGY CASE #: 092209-0001

**TYPE OF
MODALITY**

Equipment: SRH-CR2

Model: CR

Date Processed: SEP 24,2009

Problem with: BAD CASE #

Comment:

Correcting file on Image gateway server ID: A

C:\DICOM\IMAGE_IN\A0787603.DCM

**TYPE IN PATIENTS
NAME OR THE
CORRECT CASE
NUMBER HERE**

Do you want to Correct this entry? (Y/N/D/Q)// y

Lookup by case number or patient name

Enter patient or case number: _

```
Select Dicom Menu Options Option: correct
  1   Correct Clinical Specialties DICOM File Entries
  2   Correct RAD-DICOM File Entries
CHOOSE 1-2: 2   Correct RAD-DICOM File Entries
```

Select one of the following:

- P Patient
- L Loop thru file
- D Specify a Date Range

MAKE A NOTE OF THE PATIENTS NAME AND DATE OF EXAM

```
Update entries by: Loop thru file
*****Processing entry*****
```

AND MODALITY

```
PATIENT: TEST,TEST
RADIOLOGY CASE #: 092209-0001
Equipment: SRH-CR2
Date Processed: SEP 24,2009
Comment:
Correcting file on Image gateway server ID: A
  C:\DICOM\IMAGE_IN\A0787603.DCM
Do you want to Correct this entry? (Y/N/D/Q)// y
Lookup by case number or patient name
```

```
SSN: 1
Model: CR
Problem with: BAD CASE #
```

```
AnzioWin - 161.223.36.1
File Edit View Diagnose Communicate Transfer Help
*****Processing entry*****

PATIENT: TEST,TEST
RADIOLOGY CASE #: 092209-0001
Equipment: SRH-CR2
Date Processed: SEP 24,2009
Comment:
Correcting file on Image gateway server ID: A
C:\DICOM\IMAGE_IN\A0787603.DCM
Do you want to Correct this entry? (Y/N/D/Q)// y
Lookup by case number or patient name
Enter patient or case number: 092209-0001
Entry not found, enter a "^" to quit.
Enter patient or case number:
This is a required response. Enter '^' to exit
Enter patient or case number: 1
Entry not found, enter a "^" to quit.
Enter patient or case number: demo,
Select RAD/NUC MED PATIENT: demo, //
  1  DEMO,NAVAJO CHEI          <A>   M 01-01-1920          NNMC 2
  2  DEMO,NAVAJO JANE        <CWAD> F  ** SENSITIVE **      NNMC 1
  3  DEMO,NAVAJO JOHN       <A>   M 07-04-1965          NNMC 4
  4  DEMO,NAVAJO SHIMA      <A>   F 01-01-1925 XXX-XX-0000 NNMC 5
CHOOSE 1-4:
AnzioWin Version 11.3j alt-M for HELP
```

SSN: 1

Model: CR

Problem with: BAD CASE #

VERIFY THE CHART NUMBER AND NAME OF PATIENT

MAKE SURE THAT YOU SELECT THE CORRECT PATIENT FROM THE LIST

NUM

Patient: DEMO,NAVAJO JANE

PID: 1

Case #	Procedure	Exam Date	Status ofExam	Imaging Loc	
1	455	CHEST 2 VIEWS PA&LAT	07/20/09	COMPLETE	RADIOLOGY NN
2	458	WRIST 3 OR MORE VIEWS	07/20/09	COMPLETE	RADIOLOGY NN
3	461	KNEE 3 VIEWS	07/20/09	COMPLETE	RADIOLOGY NN
4	463	ANKLE 3 OR MORE VIEWS	07/20/09	COMPLETE	RADIOLOGY NN
5	592	CHEST SINGLE VIEW	09/05/07	COMPLETE	RADIOLOGY NN
6	582	CHEST 2 VIEWS PA&LAT	09/05/07	COMPLETE	RADIOLOGY NN
7	412	SPINE THORACIC AP&LAT&SWIM V	10/04/06	COMPLETE	RADIOLOGY NN
8	73	BILAT EXTREMITY VENOUS DUPL	10/02/06	COMPLETE	RADIOLOGY NN
9	594	KNEE 3 VIEWS	09/29/06	COMPLETE	RADIOLOGY NN
10	593	CT HEAD W/O CONT	09/29/06	COMPLETE	RADIOLOGY NN

'i' next to a case number denotes images collected on study.

Select an exam: _

**CHOOSE THE EXAM THAT MATCHES THE
PATIENTS EXAM THAT WAS DONE**

Patient: DEMO,NAVAJO JANE

PID: 1

Case #	Procedure	Exam Date	Status ofExam	Imaging Loc
1	455 CHEST 2 VIEWS PA&LAT	07/20/09	COMPLETE	RADIOLOGY NN
2	458 WRIST 3 OR MORE VIEWS	07/20/09	COMPLETE	RADIOLOGY NN
3	461 KNEE 3 VIEWS	07/20/09	COMPLETE	RADIOLOGY NN
4	463 ANKLE 3 OR MORE VIEWS	07/20/09	COMPLETE	RADIOLOGY NN
5	592 CHEST SINGLE VIEW	09/05/07	COMPLETE	RADIOLOGY NN
6	582 CHEST 2 VIEWS PA&LAT	09/05/07	COMPLETE	RADIOLOGY NN
7	412 SPINE THORACIC AP&LAT&SWIM V	10/04/06	COMPLETE	RADIOLOGY NN
8	73 BILAT EXTREMITY VENOUS DUPL	10/02/06	COMPLETE	RADIOLOGY NN
9	594 KNEE 3 VIEWS	09/29/06	COMPLETE	RADIOLOGY NN
10	593 CT HEAD W/O CONT	09/29/06	COMPLETE	RADIOLOGY NN

'i' next to a case number denotes images collected on study.

Select an exam: 1

****Please review the following: ****

Previous name: TEST,TEST

New name: DEMO,NAVAJO JANE

Previous ssn: 1

New ssn: 1

Previous case #: 092209-0001

New case #: 072009-455

Patient names do not match. Update? (Y/N/D/Q)//

AnzioWin

Version 11.3j

alt-M for HELP

NUM

**DOUBLE CHECK THAT
THESE MATCH**

**IF THEY MATCH THEN SAY YES TO
UPDATE**

6	582	CHEST 2 VIEWS PA&LAT	09/05/07	COMPLETE	RADIOLOGY NN
7	412	SPINE THORACIC AP&LAT&SWIM V	10/04/06	COMPLETE	RADIOLOGY NN
8	73	BILAT EXTREMITY VENOUS DUPL	10/02/06	COMPLETE	RADIOLOGY NN
9	594	KNEE 3 VIEWS	09/29/06	COMPLETE	RADIOLOGY NN
10	593	CT HEAD W/O CONT	09/29/06	COMPLETE	RADIOLOGY NN

'i' next to a case number denotes images collected on study.

Select an exam: 1

****Please review the following: ****

Previous name: TEST,TEST

New name: DEMO,NAVAJO JANE

Previous ssn: 1

New ssn: 1

Previous case #: 092209-0001

New case #: 072009-455

Patient names do not match. Update? (Y/N/D/Q)// y

Will change the following:

****Please review the following: ****

Previous name: TEST,TEST

New name: DEMO,NAVAJO JANE

Previous ssn: 1

New ssn: 1

Previous case #: 092209-0001

New case #: 072009-455

Are you sure you want to correct this entry? ? No// y_

IT WILL ASK YOU AGAIN IF YOU ARE SURE THAT THIS IS THE CORRECT INFORMATION

THEY MUST MATCH EXACTLY

SAY YES AGAIN HERE

```
AnzioWin - 161.223.36.1
File Edit View Diagnose Communicate Transfer Help

Previous name: TEST,TEST
New name: DEMO,NAVAJO JANE
Previous ssn: 1
New ssn: 1
Previous case #: 092209-0001
New case #: 072009-455
Are you sure you want to correct this entry? ? No// y (Yes)
Updating the file....
*****Processing entry*****

PATIENT: TEST,TEST
RADIOLOGY CASE #: 092309-0001
Equipment: SRH-CR3
Date Processed: SEP 24,2009
Comment:
Correcting file on Image gateway server ID: A
C:\DICOM\IMAGE_IN\A0787506.DCM
Do you want to Correct this entry? (Y/N/D/Q/P)// d

*****
*** Will log in error log (file 2006.599). ****
Please enter a reason for deleting.
For example: TEST PATIENT
Reason for deletion:
AnzioWin Version 11.3j alt-M for HELP NUM
```

IT SHOULD NOW SAY
UPDATING FILE

NEXT PATIENT IN QUEUE

SINCE WE DON'T KNOW
WHO THIS PATIENT IS
WE CAN DELETE IT

MAKE SURE YOU PUT IN
THE REASON THAT YOU
DELETED THE PATIENT

YOU ARE NOW DONE

ANY QUESTIONS?